

CASCO Pet Veterinary Clinical Housing Care

CLEANING

Through the elimination of bars, we have created housing that has easy wipe surfaces, leading to a cleaner and more hygienic environment. Most standard veterinary disinfectants are safe to use on our enclosures. If you choose to use steam cleaners, please note this should not be used on any of the framework or handles as this may cause water damage that will not be covered under warranty.

Due to the weight and construction design, moving the units should only be carried out by our expert installation team. Should you need to move your units, please contact our Helpdesk team using the details at the bottom of this document.

ADJUSTING YOUR ENCLOSURE DOORS

Over time the doors on your CASCO Veterinary enclosures may need adjusting to ensure they sit in place and lock correctly. CASCO Pet recommends checking and adjusting your doors once a month to ensure they function correctly. To make these adjustments, please follow the following simple steps:

You will need the Torque wrench supplied with these enclosures with hex fitting 4.



INSTRUCTIONS

First, open the doors 90 degrees so you can tighten / adjust the bolts securing the glass doors to the hinges.

Bolts on the enclosure side (glass with green background)


These bolts simply need to be checked to ensure they are tight and very rarely need adjusting.

1. Insert hex fitting of torque wrench into the bolt.
2. Turn the wrench clockwise until the wrench clicks.
3. This will ensure the correct tightness has been achieved.



Bolts on the door side

1. Insert the hex fitting of torque wrench into the bolt.
2. Turn torque wrench anti-clockwise to loosen the bolts very slightly. Do not remove the bolts.
3. Lift the end of the door up very slightly
4. Turn the wrench clockwise until it clicks, this will ensure the correct tightness has been achieved
5. Gently close the door to ensure it closes correctly and the glass does not touch the base of the enclosure
6. If the door is rubbing the enclosure, please repeat steps 1-5
7. Ensure the handle locates into the top of the enclosure to securely lock the door, if the door can be easily opened by pulling the handle without twisting it, the door is too low and will need raising slightly.

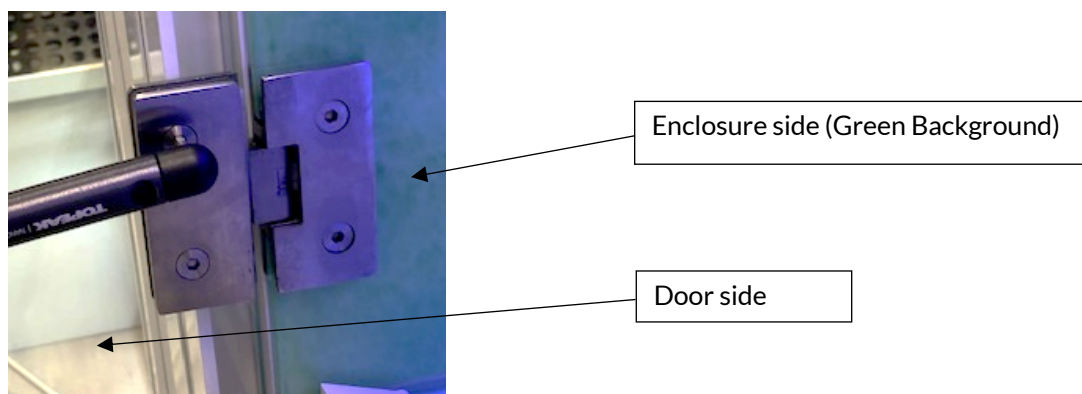

If you need any further assistance or advise, please contact our helpdesk team on helpdesk@cascopet.com or call your local CASCO Pet Office.

UK OFFICE

CASCO Pet

Unit 23-25 Lawson Hunt Industrial Park
Broadbridge Heath, Horsham, West Sussex
RH12 3JR
United Kingdom

Phone: 08450 609960

Email: uk@cascopet.com

US OFFICE

CASCO Pet

1240 E Belmont Street
Ontario, California
91761
United States of America

Phone: 866 584 522

Email: usa@cascopet.com

AUSTRALIAN OFFICE

CASCO Pet

81 Airds Road
Minto 2566
Australia

Phone: 0452 515 065

Email: au@cascopet.com

GERMAN PRODUCTION AND OFFICE

CASCO Pet GmbH

Büttervenn 22
33758
Schloß Holte-Stukenbrock
Deutschland

Phone: +49 5207 921596

Email: de@cascopet.com

IBERIAN PRODUCTION AND OFFICE

CASCO Aquastyl

EN106, Pav. 2 e 3
4620-276 Lustosa
Lousada
Portugal

Phone: 255 881 801

Email: pt@cascopet.com

FRENCH CONTACT

Phone: 0810 224 540

Email: fr@cascopet.com